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Owner's Manual

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IDENTIFICATION OF EQUIPMENT22

Identification22

Principles and bases applied to the functioning of the product.....23

Description of the Equipment23

Indication of Equipment23

MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION24

TECHNICAL SPECIFICATIONS25

- Technical characteristics25

- List of pieces and circuit scheme26

- Contents of the accessible and inaccessible markings26

- Symbologies of packaging27

- Symbologies of product27

OPERATION OF EQUIPMENT28

PRECAUTIONS, RESTRICTIONS AND WARNINGS30

- Conditions of transport and storage31

- Sensitiveness to environmental conditions foreseeable in normal situations of use .31

- Precautions and warnings “during the installation” of equipment.....31

- Recommendations for preserving the equipment.....32

- Precautions and warnings “during the utilization” of equipment32

- Precautions and warnings “after” the use of equipment32

- Precautions and warnings during the “cleaning and disinfection” of equipment.....33

- Precautions in case of alteration in the functioning of equipment.....33

- Recautions to be adopted against foreseeable or uncommon risks,
related to the deactivation and abandoning of equipment.....33

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION.....33

- Additional procedures for reuse33

- Cleaning33

- Disinfection34

- Preventive maintenance35

- Corrective maintenance.....35

UNFORESEEN EVENTS – SOLUTION OF PROBLEMS35

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Technical Name: Water Distiller

Brand: Gnatus

Trade name: Bio Aqua Water Distiller

Manufacturer/ Distributor:

GNATUS - EQUIPAMENTOS MÉDICO-ODONTOLÓGICOS LTDA.
Rod. Abrão Assed , Km 53+450m - Cx. Postal 782 CEP 14097-500
Ribeirão Preto - S.P. - Brasil

Fone +55 (16) 2102-5000 - Fax +55 (16) 2102-5001
C.N.P.J. 48.015.119/0001-64 - Insc. Est. 582.329.957.115
www.gnatus.com.br - gnatus@gnatus.com.br

Technical Duties: Gilberto Henrique Canesin Nomelini
CREA-SP: 0600891412

ATTENTION

For greater safety:

- Read and understand all the instructions contained in these Instructions for Use before installing or operating this Equipment.

Note: These Instructions for Use must be read by all the operators of this Equipment.

IDENTIFICATION OF THE EQUIPMENT

Dear Customer

Congratulations. You have made a good choice when you decided to buy a GNATUS QUALITY product comparable to the best products available in the World. This manual is a general presentation of your product and it will give you important details to help you to solve possible problems.

Please, read it and keep this with you.

Identification

Technical Name: Water Distiller

Trade name: Bio Aqua Water Distiller

Brand: GNATUS

IDENTIFICATION OF THE EQUIPMENT

Principles and bases applied to the functioning of the product

As in the water cycle in nature, the cycle begins when the sunbeams warm waters of the seas and rivers forming water steam that rises into the air (evaporation) when the vapor cools down clouds are formed (condensation) and when they become heavy they eliminate water back through the rain (precipitation).

In Bio Aqua water Distiller, water is boiled in the boiling chamber and when it reaches a temperature of 100°C, bacteria, viruses and other organisms that may be present are killed. Boiling water turns into steam, eliminating dissolved solids, chemical toxins, heavy metals and other contaminants. The steam rises through the condensation coil, it is cooled and condenses into drops of pure water. After this process, the water still passes through an activated charcoal filter (optional), eliminating odors and flavors. Pure water is collected in a plastic bottle provided with the product.

Description of Equipment

The portable Bio Aqua water distiller totally eliminates the total dissolved solids (minerals, heavy metals, toxins, or any other contaminant).

- It has high capacity "4Liters": sufficient volume for many different uses;
- Purify water up to 99.9%, meeting all required standards;
- It is portable and easy to install;
- Robust and easy to clean, entirely thermoplastic-covered cylindrical body;
- Internal stainless steel vat: durability and ease of cleaning;
- Manual supply, it does not require hydraulic installations;
- It has automatic power-off function after each distillation cycle

Indication of Equipment

The Bio Aqua water Distiller is indicated for use in autoclaves, laboratories, compounding pharmacies and all procedures that require use of distilled water.

It should be used and handled by qualified person (professional personnel properly regulated according to the local legislation of the country) following the instructions in this manual.

It is the user's obligation to use only the device in perfect conditions and protect him/herself, patients and third parties from eventual dangers.

Note

ACTIVATED COAL FILTER:

- When distilled water is indicated for human consumption, shall be purchased the activated coal filter therefore water has a much more good fresh taste (optional item available for sale).

MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION


The content of this page is for information purposes, and the device may be presented different from that illustrated. Therefore, when purchasing the product check the technical compatibility between the device, coupling and accessories.

The drawing illustrates the device with all the optional items, your device will consist only of the items chosen during your purchase option.

The use of any part, accessory or material not specified or referred to in these instructions for use is the user's responsibility.

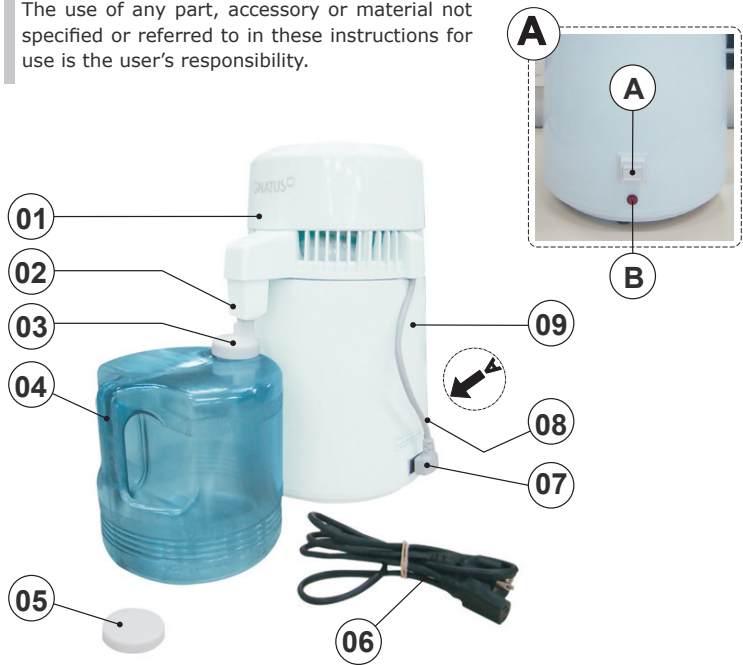- 01 - Condensation cover
- 02 - Nozzle guide of the water condensation cover
- 03 - Nozzle guide of the water Reservoir
- 04 - Reservoir for collecting distilled water
- 05 - Reservoir cover for collecting distilled water
- 06 - Power input cable
- 07 - Condensation cover's power plug
- 08 - Power input cable
- 09 - Internal stainless steel vat
- A - Power button
- B - Indicator Led (Power button)

TECHNICAL SPECIFICATIONS

Device's Technical Features and Accessories

General

Model:

Bio Aqua water Distiller

Operation mode: Continuous operation with intermittent load on.: from 40 to 200min - off.:30min.

Power Supply

Power supply voltage:

127V~:100V~ - 140V~ / 220V~:198V~ - 242V~

Frequency:

50/60 Hz

No of phases:

Monophase / Biphase

Power:

750W

Other Specifications

Distilled volume:

1,2L/h

Evaporation chamber capacity:

4L

Reservoir capacity (plastic galloon):

3,5L

Evaporation chamber internal material:

Stainless steel

External housing:

Thermoplastic material

Heat efficiency:

>95%

Filter (optional):

Activated charcoal filter

TECHNICAL SPECIFICATIONS

Device's Technical Features and Accessories

Other Specifications

Dimensions of the product:
380 x 235 mm
Dimensions of the packing:
450 x 260 x 290 mm
Net weight:
3,5 kg
Gross weight:
4,7 kg

The materials used to produce the equipment are Biocompatible.

List of pieces and circuit scheme

Gnatus Company declares that the supply of the circuit scheme, list of pieces or any other information that propitiate technical attendance for the user, can be request if there is an agreement between the user and Gnatus Company.

Content of accessible and non-accessible demarcations

FABRICANTE E RESPONSÁVEL PELA GARANTIA MANUFACTURER AND RESPONSIBLE FOR THE WARRANTY FABRICANTE E RESPONSÁVEL PELA GARANTIA

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TECHNICAL SPECIFICATIONS

Packing symbols

It determines the maximum quantity of boxes which can be stacked during transportation and storage "as per packaging".

Packing to be transported and / or stored avoiding humidity, rains and wet floor.


Packing to be transported and / or stored with the harrows up.

The packing must be stored and transported away from direct sun light exposure.

Packing to be transported and / or stored with care (should not suffer drop and neither receive impact).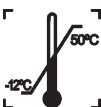

Temperature limit for the packing to be stored or transported.

Product symbols

Careful : It indicates an important instruction for the operation of the product. Not following it can cause dangerous malfunctioning.

Power button

Note: It indicates useful information for operation of the product.

Important: It indicates an instruction of safety for operation of the product. Not following it, can lead to serious danger to the patient.

Landing (in many parts of the equipment) indicates the condition of being landed.

Warning - see the manual


OPERATION OF EQUIPMENT

General use:

The distiller consists of two main parts: the condensation cover and evaporation chamber.

The cover contains a condensing coil, a mini fan and the activated charcoal filter. The evaporation chamber is made of stainless steel, with an electric heating system on the bottom part.

A - Remove the condensation cover, rinse the stainless steel chamber, then fill the chamber with tap water until the maximum range that is approximately 4 liters (do not add water above this range);

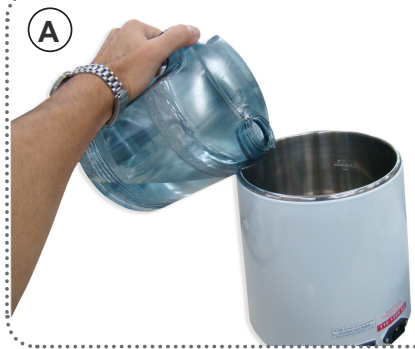If the vat's supply is made up to the maximum level indicated on the side of the vat "4L," the galloon having its maximum capacity of "3.5 L" will overflow before completing all distillation.

B - Attach the condensation cover and connect the cover cable to the plug in the device's base unit.

C - Put the plastic bottle directly under the guiding water nozzle;

D - Connect the power supply cable directly into an appropriate power plug;


OPERATION OF EQUIPMENT

E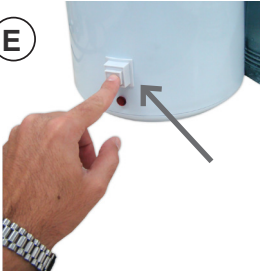

E - The distiller will automatically turn on in the first use, when this does not occur use the power button switch "SPSP".

The time to produce 4 liters of distilled water is approximately 4 hours. The distiller will automatically turn off when all the water is evaporated from the chamber.


Activated coal filter installation

Only will be necessary to use the activated coal filter if water is indicated for human consumption

A

B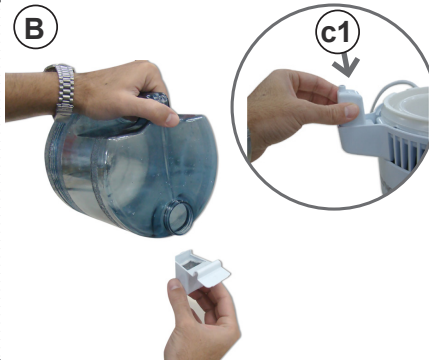

- A - Remove the nozzle guide located in the condensation cover ;
 B - Prepare the activated coal filter to replace the nozzle guide, put 2 to 3 glass of water in the filter before each cycle of distillation. After the filter is dampened fit it in the condensation cover (C1).

Avoid connecting the apparatus without water in the camera.

After a complete distillation cycle, do not open the condensation cover immediately. Wait for the cover cools down for at least 20 minutes.

When using the distiller for the first time, discard the first cycle of distilled water, so that the internal components are completely sterilized by this first cycle.

OPERATION OF EQUIPMENT

Quantity of added water	1L	2L	3L	4L
Heating time	8-12 min.	15-20 min.	20-25 min.	30-40 min.
End time	40-60 min.	80-100 min.	130-160 min.	170-200 min.

 **Suggestion: Use hot water in order to reduce the process time.**

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Never cover or block the device’s ventilation openings.

Before each distillation cycle, the activated charcoal filter shall be moistened.

- The filter shall be replaced every three months.

After a complete distillation cycle, do not open the condensation cover immediately.

Wait for the cover to cool down for at least 20 minutes.

- Never touch the internal surface of the tank when the device is in use. Wait for the total cooling, (approximately 30 minutes).

Improper use may result in burns.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Transportation and storage

This equipment must be transported and stored observing the following directions:

- Avoid falls and impacts;
- Keep it dry, do not expose it to rain, water drops or wet floor;
- Keep it away from water and direct sunlight, and in its original wrapping;
- Don't move it over irregular surfaces, protect it from rain and observe the maximum stack quantity specified in the packaging;
- Transportation and storage temperature range: -12°C to 50°C;
- Transportation and storage relative humidity range: 0°C to 90°C;

The Equipment maintains its condition of safety and efficacy, provided that it is maintained (stored) as mentioned in this instruction of use. Thus, the equipment will not lose or alter its physical and dimensional features.

Sensitivity to environmental conditions in normal situations of use

- The equipment has been planned not to be sensitive to interference such as magnetic fields, external electrical factors, electrostatic discharge, pressure or variance of pressure, provided that the equipment is installed, maintained, clean, preserved, transported and operated as per this instruction for use.

Precautions and warnings “during the installation” of equipment

- Check the voltage of the equipment at the moment of executing the electrical installation.
- Position the equipment in a place where it will not get wet.
- Install the equipment in a place where it will not be damaged by pressure, temperature, humidity, direct sunlight, dust, salts, or sulfur compounds.
- The unit should not be submitted to inclination, excessive vibrations, or blows (including during transportation and handling).
- This equipment was not planned for use in an environment where vapors, anesthetic mixtures inflammable with air, or oxygen and nitrous oxide can be detected.
- Before the first use and/or after long interruptions from work such as vacations, clean and disinfect the equipment..

Grounding:

Grounding is compulsory. Only in this way the user is in total safety. It is only the user's responsibility for not grounding, implying in warranty loss.
Use three-prong plug. Never connect the grounding to the neutral.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Recommendations for the dental equipment maintenance

Your Gnatus equipment has been designed and developed according to the standards of modern technology. Similarly to other kinds of equipment, it requires special care, which is many times neglected due to several reasons and circumstances.

Therefore, here are some important reminders for your daily routine. Try to follow these simple rules, which will save you a lot of time and will avoid unnecessary expenses once they start making part of your working procedure.

Precautions and warnings “during the use” of equipment

- The device shall be operated only by qualified personnel properly authorized and trained (Surgeons Dentists and Qualified Professionals)

- In the need of maintenance, use only the services of Gnatus Authorized Technical Assistance.

- The device has been manufactured to support continuous and intermittent operation, so follow the cycles described in these Instructions for Use.

- Although this device has been designed according to electromagnetic compatibility standards it may, under extreme conditions, cause interference with other devices. Do not use this device with other devices that are very sensitive to interference or with devices that create high electromagnetic disturbance.

- Do not subject plastic parts to contact with chemicals used in the dental treatment routine. Such as acids, mercury, liquid acrylics, amalgams, etc..

- Never block the device's ventilation openings. Keep openings free from lint, hair and other materials.

- Do not fill the tank above the maximum line to avoid overflowing.

- Never touch the internal surface of the tank when the device is in use.

- Wait for the total cooling (approximately 30 minutes). Misuse may result in burns.

- Before each distillation cycle, the activated charcoal filter shall be moistened.

Gnatus shall not be responsible for:

- Use of the equipment differing from that for which it is intended.

- Damages caused to the equipment, the professional and/or the patient by the incorrect installation and erroneous procedures of maintenance, differing from those described in these Instructions for use which come with the equipment or by the incorrect operation of it.

Precautions and warnings “after” the use of equipment

- Turn off the main switch of the dental set when it is not in use for an extended period of time.

- Always maintain the equipment clean for the next operation.

- Do not modify any part of the equipment. Do not disconnect the cable or other connections without need.

- After using the equipment, clean and disinfect all the parts which may be in contact with the patient.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Precautions and warnings during the “cleaning and disinfection” of equipment

- Before cleaning the equipment, turn off the main switch.
- Avoid spilling water, even accidentally, or other liquids inside the equipment, which could cause short circuits.
- Do not use microabrasive material or steel wool when cleaning, or employ organic solvents or detergents which contain solvents such as ether, stain remover, gasoline etc.

Precautions in case of alteration in the functioning of equipment

- If the equipment has any abnormality, check if the problem is related to any item listed in the topic of unforeseen events (failures, causes and solutions). If it is not possible to resolve the problem, turn off the equipment, remove the power supply cable from the socket and contact your representative (Gnatus).

Precautions to be adopted against foreseeable or uncommon risks, related to the deactivation and abandoning of equipment

In order to avoid environmental contamination or undue use of the Equipment after it has become useless, it should be discarded in the suitable place (as per the local legislation of the country).

- Pay attention to the local legislation of the country for the conditions of installation and disposal of residue.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Additional procedures for reuse

The equipment can be reused in undetermined, i.e. unlimited, quantities, only needing to be cleaned and disinfected.

Cleaning

Important: In order to execute cleaning or any type of maintenance, ensure that the equipment is disconnected from the electrical network.

Internal cleaning of the tank:

NOTE: Changing in visual appearance inner the destiladora tank is a normal feature of the product due to the heating of it.

- Spots and oxidations signs may occur due to the continuous use and it is totally or partially removed according to the recommended procedure as below:

In order to execute the internal cleaning of your destiladora we recommend using the product “Steel-Clean”. Product notified at ANVISA/MS.

WARNING: Perform cleaning according to specified by the manufacturer (see the packing label). In the case of incorrect cleaning procedure it may occur a darkening in the surface.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Product Composition: Surfactant anionic, inorganic acid, sequestrant, preservative, dye and vehicle.

WARNING: After the cleaning procedure perform a complete cycle of distilling and then eliminate the water of the reservoir and dry the tub using a clean cloth.

 **NOTE:** The registration at the Ministry of Health of "Steel-Clean" is performed separately from the product described in this manual, as the "Steel-Clean" is not manufactured by Gnatus.

External cleaning:

 The cleaning procedure should be executed at the start of the working day and after each patient. Always turn the main switch off before executing the procedures of daily maintenance.

To clean the equipment, we recommend the use of "BactSpray (Reg nº MS: 3.2079.0041.001-5) or any other similar product:

Active component: Benzalkonium chloride (tri-quaternary ammonium)

Solution 50%..... 0.329%

Chemical composition: Butyl Glycol, Decyl polyglucose, Sodium Benzoate, Sodium Nitrate, Essence, Deodorized Propane / Butane, demineralized Water.


For more information concerning cleaning procedures, see manufacturer's instructions.

WARNING:

- In order to prevent risks and damages to equipment, make sure that the liquid does not enter into the unit.
- The application of other solvent-based cleaning products or sodium hypochloride isn't recommended, because they may damage the equipment.

 **NOTE:** The registration at the Ministry of Health of the "BactSpray" is executed separately from the product described in this manual, as the "BactSpray" is not manufactured by Gnatus.

 **NOTE:** Use gloves and other systems of protection, during the disinfection.

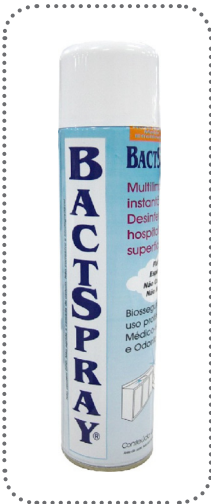Disinfection

- Use clean and soft cloth dampened in alcohol 70% to disinfection of the equipment.
- Never use corrosive disinfectants or solvents.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Preventive Maintenance

In order to protect your equipment, seek Gnatus technical assistance for periodic revisions of preventive maintenance.

Corrective Maintenance

If the equipment has any abnormality, check if the problem is related to any of the items listed in the item Unforeseen Events (situation, cause and solution).

If it is not possible to solve the problem, turn off the equipment, and request Gnatus technical assistance.

UNFORESEEN EVENTS – SOLUTION OF PROBLEMS

 Upon coming across any problem in operation, follow the instructions below to check and repair the problem, and/or get in touch with your representative.

Problem	Probable cause	Solution
-Completely out of order	-P l u g disconnected -Lack of Power supply. -Condensation cover's plug disconnected to the device. -"SPSP" key not activated.	-Connect the plug. -Wait for normalization. -Connect the plug. -Press the SPSP power button again.
-After a distillation cycle, a low production of distilled water was obtained.	-Low level of water in the vat. - Condensation cover is detached to the evaporation chamber.	-Regulate the water level. -Couple the condensation cover.
-Steam leaking.	-Condensation cover is detached to the evaporation chamber. -Damaged Seal ring. -Damaged Cover set: Condensation coil or mini fan.	-Couple the condensation cover. -Contact the Gnatus technical assistance. - Contact the Gnatus technical assistance.

WARRANTY OF EQUIPMENT

This equipment is covered by the warranty terms counting from the date of installation, as specified below; provided that the defect has occurred in normal conditions of use and that the equipment has not remained stored for more than 06 months counting from the issue date of the sales document until the date of the actual installation.

- WARRANTY TERMS: Verify the guarantee certificate;

- LOSS OF THE WARRANTY:

- A) Attempt to repair using an inadequate tool or by unauthorized technicians;

- B) Installation of the equipment by an unauthorized technician;

- C) Damage arising from inappropriate storage or signs of infringement;

- D) Incorrect use of the equipment;

- E) Use of a cleaning product not indicated by the factory;

- F) Falls or blows which the equipment may undergo or lack of observation of an compliance with the guidelines of the Owner's Manual. Repair or replacement of parts during the warranty period shall not extend the validity term of their warranty.

- This warranty does not exempt the customer from paying the service charge for the visit and the travel expenses of the technician, except when the customer sends the equipment to execute the maintenance inside the establishment of the technical assistance.

- "Consumer Defense Code - art. 50, unique paragraph".

- The Warranty Certificate comes with the product and must be filled in upon the date of installation by the Gnatus Authorized Technician.

- Queries and information: GNATUS Help Desk (+55) 16 2102-5000.

FINAL CONSIDERATIONS

The most important aspect related to equipment care is that concerning spare parts.

To guarantee the life span of your equipment, use only original Gnatus spare parts. They are sure to follow the technical specifications and standards required by Gnatus.

We must also point out to you our chain of authorized dealers. Only dealers that make part of this chain will be able to keep your equipment constantly new for they count on technical assistants who have been trained and on specific tools for the correct maintenance of your equipment.

Doubts and information: GNATUS Call center (55-16) 2102-5000.

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